Dräger Alco-Check 0,5

DEUTSCH

Allgemeines

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ENGLISH

General

The alcohol content in exhaled air is dependent on the alcohol content in the blood. If the Alco-Check reveals alcohol in the exhaled air, a subsequent blood-alcohol test or another approved method is essential to establish a legal assessment. No blood-alcohol test is of course necessary when the Alco-Check produces no indication.

**Alco-Check contains corrosive substances.
Keep out of the reach of unauthorized persons.**

Prerequisites

Testing of the exhaled air for alcohol must be performed no earlier than 15 minutes after

- the consumption of alcoholic or aromatic beverages (e.g. fruit juice),
- use of mouth sprays,
- smoking.

Otherwise an excessively high value is measured.

Alco-Check contains a yellow reaction layer which turns green when exposed to alcohol vapour.

An unused Alco-Check which has turned green before testing must not be used.

An Alco-Check which has been opened but not used, or which may have not changed colour during a test, must not be used again. The unit may be used for only one test.

Measurement

- Remove the Alco-Check from the packaging and bend the ends one after another (see figure A) until they click. The Alco-Check is then activated and can be used.
- Insert the Alco-Check into the opening of the flattened measuring bag with the printed arrow pointing towards the bag (B).
- Inhale deeply and blow strongly into the Alco-Check input, filling the bag completely without pausing to inhale again. The blowing should be finished in 15 seconds to 20 seconds.

Evaluation

With a blood-alcohol concentration between 0.3‰ and 0.4‰, the green colour does not reach the indicator line (C).

If the green colour reaches or goes further than the indicator line, a blood-alcohol content of 0.5‰ or more can be assumed.

Disposal

- Alco-Check contains corrosive substances, and the two ends should therefore be sealed off with the plugs supplied after use (D).

Do not simply throw away used Alco-Check or Alco-Check which have reached the end of their shelf life. Try to return the tube to place of purchase. If this is not possible, then please dispose with other hazardous waste.

Liability

Alco-Check can only be used for guideline measurements.

We therefore accept no liability whatsoever.

Even when used as carefully as possible, there may be deviations between the breath-alcohol concentration and the blood-alcohol concentration which is the sole decisive factor in law.

Our advice:

Even blood-alcohol concentrations less than 0.5‰ can affect driving ability.
Do not drive under the influence of alcohol.

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